


Limbo®

Waterproof protectors

Waterproof Protection for casts, dressings & PICC lines

- 100% Waterproof
- Easy to use. (instructions enclosed)
- Tough, durable construction
- Comfortable and non-constrictive neoprene seal
- 100% Latex free
- A full range of sizes available
- Made in Great Britain

Sizing Guide

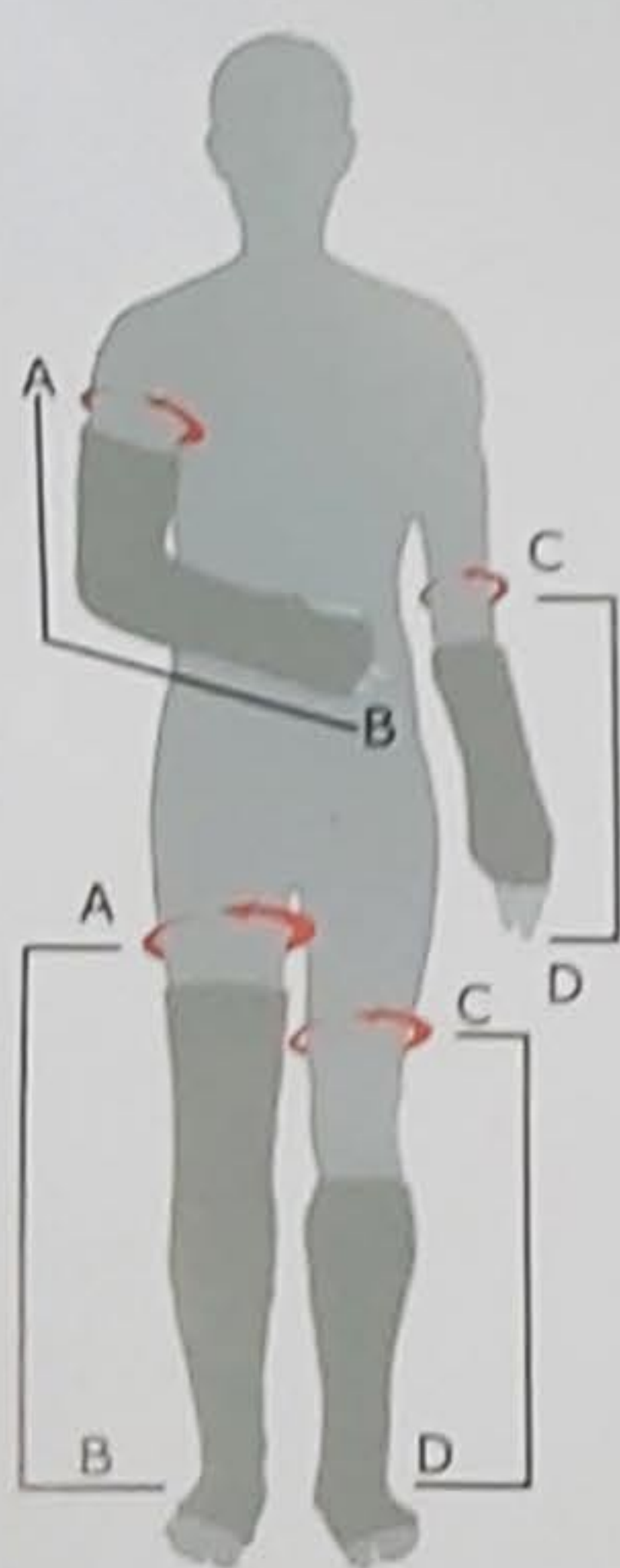

Arms

For Full Arms

Measure circumference of upper arm as indicated by the red arrow at point A and length from A to B.

For Half Arms

Measure circumference of arm above the elbow as indicated by the red arrow at point C and length from C to D.

Please note:

For full arm casts it is helpful to measure the circumference of the elbow if the cast is particularly large.

Legs

For Full Legs

Measure circumference of leg as indicated by the red arrow at point A and length from A to B.

For Half Legs

Measure circumference of leg above the knee as indicated by the red arrow at point C and length from C to D.

Please note:

If the cast is particularly large it will be helpful to measure the circumference at the widest point. For very slim legs we will need the patient's shoe size.

Full
Arm

Half
Arm
(seals
above
elbow)

Full
Leg

Half
Leg
(seals
above
knee)

See the label on the front of this bag for model and sizing details

Notes and Safety Warnings

- The Limbo should be used according to the enclosed instructions.
- Never use the Limbo for anything other than its intended purpose as a cast or dressing protector.
- Never use the Limbo as a flotation device.
- Do not attempt to use the Limbo in recreational water activities.
- Children or persons with special needs using the Limbo should be supervised at all times.
- The Limbo is a non-sterile product and should not be used on open or undressed wounds or when any danger of infection is present.
- If you intend to use your Limbo while under medical care, always obtain approval from your medical provider.

WARNING
THIS BAG IS NOT A TOY.
TO AVOID DANGER OF
SUFFOCATION KEEP
OUT OF REACH OF
BABIES AND CHILDREN.

Exclusively distributed in Australia by

HealthSaver Pty Ltd
Innovative Healthcare Products

For more information or to see our full model range, contact:
14/140 Wecker Road, Mansfield, QLD 4122, Australia.
Phone: 1300 767 888 Fax: 1300 767 999
Email: info@healthsaver.com.au
www.healthsaver.com.au