

Tyrex Measuring Guide

Shop online ↘

acmobility.com.au



Copyright © AC MOBILITY PTY LTD


Measuring for a Tyrex ramp is easy! All you need is a tape measure



A. Height

Measure the height of the door threshold or step where the ramp will be going. To ensure the ground is level, make sure you measure the left and right sides of the threshold.

Tyrex ramps can be custom made to cater for uneven surfaces.

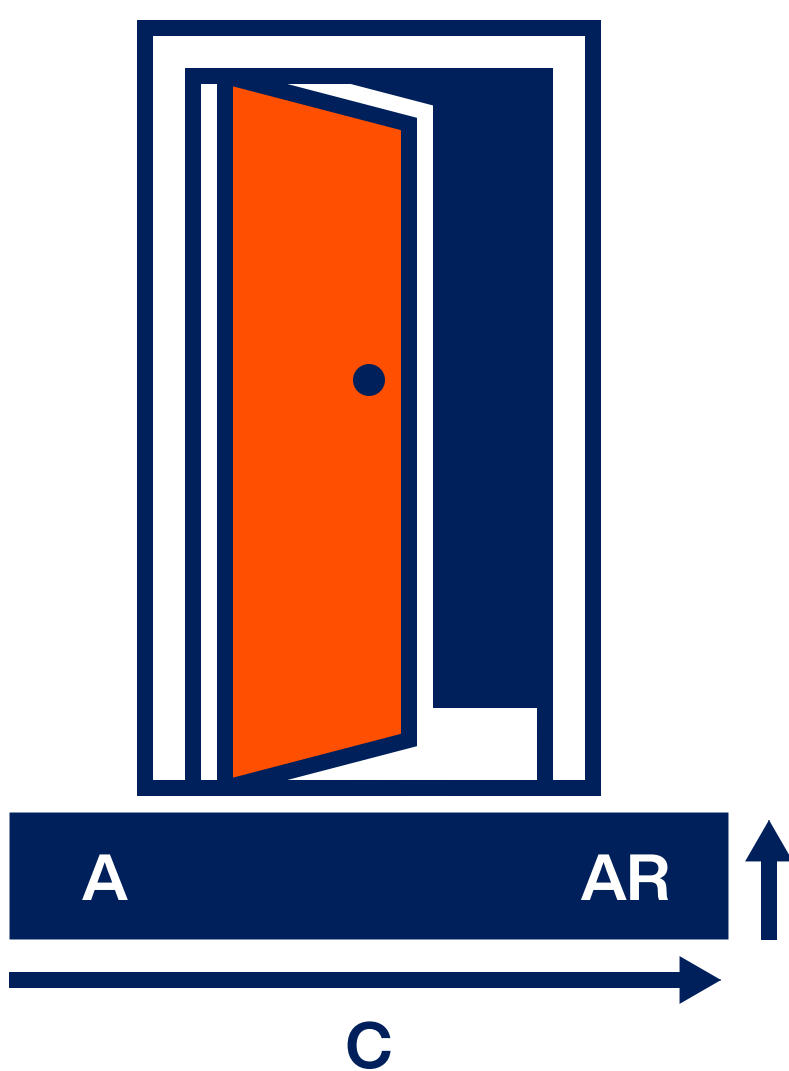


B. Length

To adhere to AS 1428.1-2009 by having a 1:8, 1:10 or 1:14 gradient slope, simply multiply the height of the ramp by 8, 10 or 14 to determine the length.

C. Width

Measure the width across the threshold or step from left to right.

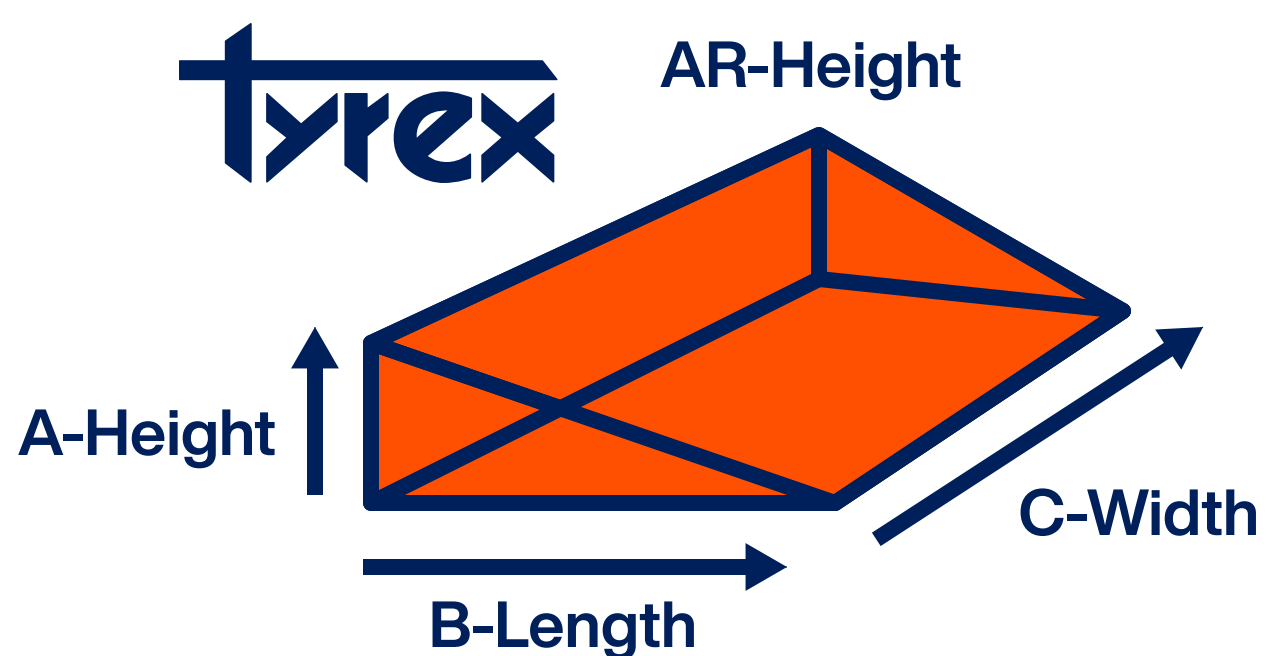

Ramp Measurements

A _____ B _____ C _____ AR* _____

* AR measurement is only required if the threshold surface is uneven.

Showroom | 5 Boulder Rd, Malaga WA 6090
Monday – Friday | 8:30am – 4:30pm
Ph | 08 9209 1777

Your Mobility, Our Focus



AC MOBILITY